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EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case. Distributor: this instruction manual must be given to the users of this product.

INTRODUCTION

The Lofoten WC lift is a medical aid device designed to assist with sitting and standing up from the toilet. Simply operate the control located on one of the armrests, and the seat will gently rise or lower. The Lofoten is placed around your existing toilet without any need for installation or modifications.

PRODUCT DESCRIPTION

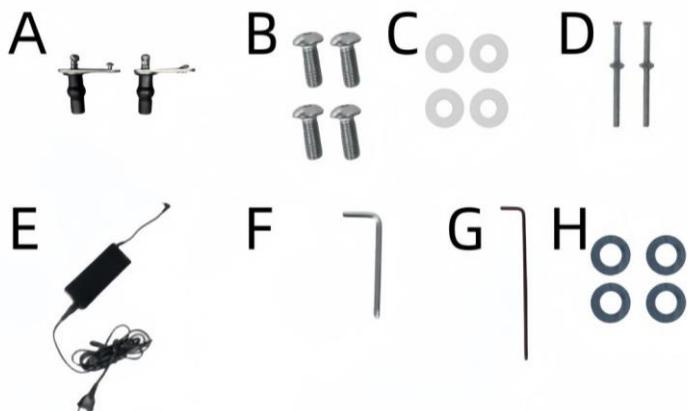
LIST OF COMPONENTS	
1	Armrests
2	Switch Button
3	Lithium Battery
4	Power Level Display
5	Remote Control
6	Alarm Button
7	Backrest
8	Bucket

ASSEMBLY AND USE

Please inspect the parts for any damage or defects before use. Only use the device if you are certain that there are no damaged or missing parts.

List of Parts:

List of parts	
A	Bolt Fixing Screws
B	Small Screws
C	Washers
D	Large Screws
E	Charger
F	Allen Key + Screwdriver
G	Round Head Allen Key
H	Metal Washers

Assembly:

Connect the right handle (switch button) to the cable located in the central frame. Ensure that the white marks on both wires match. Then, insert the left handle into the other hole of the central frame.

Tighten the handles with the small screws. Place a plastic washer and a metal washer (H) between the tube and the screw, then use the Allen key (G) to tighten securely.

Position the armrest (1) onto the handles, making sure to press down until you hear a satisfying click, indicating that it is properly engaged.

For added safety, it is imperative to secure the armrests firmly. To do this, use the large screws provided (D), along with a small nut. With the help of the Allen key and screwdriver (F), tighten the screws firmly to ensure stability and durability.

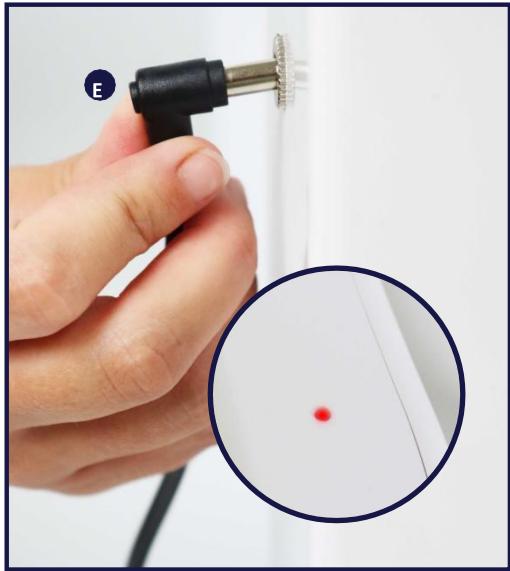
The feet have been set to their minimum height. To proceed with the adjustment:

1. Rotate the feet counterclockwise.
2. When you reach the desired height, secure the nut by turning it until it is properly positioned at the top of the foot.
3. Ensure that it is firmly and securely fixed in place.

Finally, connect the charger (E) to the frame and plug the adapter into an electrical outlet. The red light on the frame indicates that the cable is properly connected and the frame is ready to be used.

To install the WC lift on your toilet, unscrew the lid and the bowl (the upper parts of the toilet), leaving only the ceramic base. Slide the structure towards the toilet, ensuring that the lower part of the inner hole of the structure is inside the toilet.

Use:

Press and hold the left button to raise the frame and assist the user in sitting on the toilet. Press and hold the right button to lower the frame and help the user sit down on the toilet. The toilet frame tilts at an angle of 33°.

If you have selected the premium model, you can use the remote control directly to adjust the movements up and down.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 200 Kg.
- Make sure that all parts are secure and in good working order.
- Use only accessories and parts provided by the manufacturer.
- Do not make any adjustments to the product while it is in use.
- In case of damaged, broken, or worn parts, contact the dealer.
- Before your first use, please read the instructions carefully to become more familiar with the WC lift and to make its use simpler and more controllable.
- Please keep these instructions for anyone who might touch or use the WC lift.
- First assess the user to ensure that the product is appropriate. Assessment factors include, but are not limited to, the user's weight, volume, cognitive abilities, joint mobility, etc. When a disabled person uses the device, a caregiver or alarm device should be nearby.
- During use, the user should sit down slowly and calmly to avoid excessive stimulation and damage to the WC lift or to individuals.
- Please install the WC lift according to the installation instructions.
- Do not use if parts are missing.

- Users with limited physical abilities should be monitored or assisted when using the device.
- The handles are designed to support standing up and/or sitting down, using both at the same time. Do not place all your body weight on a single handle.
- Do not stand on this product. It is not a climbing device.
- Do not place 100% of your weight on a single armrest; instead, distribute your body weight evenly on both armrests. Overloading may cause the armrest to break.
- Check that all components are correctly assembled and screwed in before use.
- Avoid placing the device near heat sources.
- Any serious incidents involving the product must be reported to the manufacturer as well as to the competent authority of the member state in which the user and patient are located.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regularly clean the lift with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, dry, soft cloth.
- Ensure that there is no accumulation of dirt or debris on the lift.
- Do not use cleaning products containing phenol or chlorine, as this may damage the aluminum and polyamide materials.
- Periodically check the wear and strength of all fastenings. Replace any broken, damaged, or worn parts immediately.
- In case of malfunction, do not attempt to repair the lift yourself. Consult an authorized dealer.
- When used by different users, please clean it thoroughly.
- Ensure that the lift is completely dry before storing it.
- The charger must be unplugged from the power outlet before cleaning.
- Do not rinse the charger with water; clean it with a dry cloth.
- For disinfection in hospital environments, follow the instructions of a hygiene specialist.
- The WC lift should be inspected or repaired by professionals every 12 months.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows

Check the item	Anytime	Weekly
Check the condition and operation of the battery	X	
Inspect the state of the feet, wheels, and handles		X
Check the operation of the brakes	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.

- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Overall Dimensions	60.6 cm (width) x 52.5 cm (depth) x 38.5 to 48.5 cm adjustable height (with feet) or 42 to 45 cm (with wheels).
Width Between Armrests	48 cm
Cutout	27.5 x 23 cm
Power Supply Voltage	100 - 240 V
Frequency	50/60 Hz
Charger Cable Length	325 cm / 128 inches
Maximum Supported Weight	200 kg
Frame, Armrests, and End Caps	ABS
Foot Structure and Handles	Stainless Steel
Handles	Rubber

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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