Protective Helmet

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CDP54

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Dear customer, thank you for placing your trust in an EMO product. By choosing our line of orthoses, you have purchased a range of quality products of the highest sanitary standard. Please read these instructions carefully. If you have any questions, please contact your doctor or specialist retailer.

REGULATIONS ^[MD]
This medical device complies with Regulation (EU) 2017/745 of the European Parliament and of the Council on medical devices and the respective national regulations. A Risk Analysis (UNE EN ISO14971) has been carried out to minimise all existing risks. Class I medical device, non-sterile, without measuring function.

IMPORTANT INFORMATION
Read all warnings and instructions before using this product. Proper application is essential for proper function and to reduce the risk of injury or re-injury inherent in the use of any product of this type.

ITEM NAME:
CDP54 BLACK PROTECTIVE HELMET S (44-50cm)
CDP54 BLACK PROTECTIVE HELMET M (50-54cm)
CDP54 BLACK PROTECTIVE HELMET L (54-60cm)

PURPOSE:
Adjustable full head protection
Avoids possible head contusions
Protects from possible scratches and abrasions

INDICATIONS:
Cerebral palsy. Patients at risk of suffering cranial contusions. Spasticity. People with epilepsy. High risk of falling in elderly people with degenerative diseases.

DESCRIPTION AND CHARACTERISTICS:
Padded head protection helmet made of breathable three-layer material. Rear opening for adjusting the cranial adjustment by means of Velcro. Chin adjustment by means of two lateral pins. Incorporates chin guard. Suitable for indoor and outdoor use.

BRACE LENGTH / SIZE SELECTION
S (44-50cm)
M (50-54cm)
L (54-60cm)
Measurement: head circumference just above the ears.

MATERIALS
Outer 100% PES
Inner
· Polychloroprene 22,6%.
· Nylon
Inside 100% PES

INSTRUCTIONS FOR USE AND FITTING:
The correct choice of size is essential to achieve maximum effectiveness of the

treatment, as well as prolonging the useful life of the orthosis. Insert the head into the helmet. The adjustment area should be at the back of the helmet. Adjust the helmet to the head using the rear Velcro. Adjust the chin strap by adjusting its tension until the helmet is securely in place. The user must regularly inspect the product to ensure that it is in perfect condition and that it is working properly. Do not use the product if it is damaged. The intervention of a third party in the fitting is not necessary except in cases of limited strength or coordination. In order for the product to maintain its compression or support properties, the orthosis must adapt to the affected area without causing discomfort or discomfort during use. Excessive compression may cause intolerance to the treatment.

IMPORTANT INFORMATION
Please read all warnings and instructions before use. Correct application is essential for the product to work properly and to reduce the risk of injury or re-injury inherent in the use of any product.

PRECAUTIONS ^[!]
If any allergic reaction, irritation, skin alteration, swelling or sensitisation occurs, remove the product and consult the prescribing physician or health care professional. Do not use directly on open or reddened scars without proper protection. Products marked with the symbol ^[!] contain natural rubber latex and may cause allergic reactions in people sensitive to latex. Do not expose to a flame or heat source at very high temperatures. The brace should fit snugly but should not compress excessively. Inspect the fit of the product to ensure that it fits properly. Do not over-tighten. Check regularly to ensure that circulation is not compromised. If in doubt as to size, always choose the larger size. The user should regularly inspect the product to ensure that it is in perfect condition and that it is working properly. Do not use the product if it is damaged.

WARNINGS
The use of the orthosis is conditional on the indications. It is recommended that the orthosis be used by one patient only. For correct fitting of the orthosis, follow the manufacturer's instructions. Follow the indications or recommendations of the prescriber or healthcare professional. For a custom fitting, consult a specialist. The specialist will provide the necessary information to ensure that the end user or the person responsible for fitting the product understands how to use the product safely. Product not suitable for sterilisation. Keep out of reach of children. For disposal of the product use the solid waste container. For disposal of the packaging, depending on the packaging material, use the paper/cardboard container or the plastic container. In all cases, the

country's regulations must be strictly complied with. Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or the patient are established.

WASHING, MAINTENANCE AND STORAGE
Wash periodically with lukewarm water (maximum 30°C), neutral soap and allow to air dry. During washing, if the garment has velcro fasteners, these should be glued together or kept closed on the garment. Do not expose to direct heat sources such as hot surfaces, direct sunlight or other sources of ignition. The orthosis should be checked to ensure that it is thoroughly dry before storage, use and re-use. Contact with ointments, creams, alcohols and solvents may deteriorate the garment and shorten its useful life. Store at room temperature, preferably in the original packaging.

QUALITY CERTIFICATES AND GUARANTEES
ESPECIALIDADES MÉDICO-ORTOPÉDICAS, S.L. states that should you detect any defect within the guarantee period, please contact your distributor so that the problem can be solved as soon as possible. The guarantee will not be valid if the product has been damaged by improper use, or if the instructions and recommendations for use explained in this brochure have not been followed. **IMPORTANT!** Please do not forget to keep (with date of purchase and stamped by the seller) and present the complete product (case and body of the product), these requirements will be essential for any claim. With regular maintenance, the product will be reliable and durable and can be reused following the indicated measures for 36 months, after this period of time, it is recommended to change to a new brace.

QUALITY CERTIFICATES:
All materials used have been tested, comply with the applicable European standards and offer maximum safety and comfort during use. Oeko-Tex Standard 100 certificate: The Oeko-Tex label guarantees the absence of substances harmful to health and certifies the absence of harmful substances in all materials from which the orthosis is made. Guarantee within the framework of the legal regulations.

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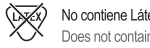
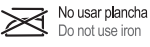
Reciclado
Recycled

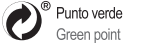
Mantener seco
Keep dry

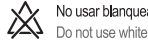
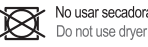
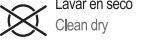
Mantener alejado de la luz
Keep away from light

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